

## How to Assemble Wrist Strap on the CMS-50F Oximeter

1). Undo the black Velcro tab and insert it through both strap holes in the oximeter with the Velcro side towards the oximeter.



2). Pull the rest of the strap through until the oximeter is against the metal ring.



3). Separate the smaller Velcro tab next to the metal ring and pull it out from the oximeter strap holes.



4. Reposition the small Velcro tab back to where it was originally attached.



5). You can now pull the Velcro tab around through the metal ring.



6). When you position the strap around your wrist you may have to pull the Velcro strap and the blue fabric through the ring to get a snug fit.

